

# TESTPACK™ PLUS STREP A WITH OBC


QUICK REFERENCE GUIDE

## Before You Begin

- Please read the package insert in its entirety, prior to use.
- Gather the materials you will need.
- Put on your disposable gloves.
- All components must be brought to room temperature (between 18°C–30°C) prior to testing.


## 1 EXTRACTION

- Add 3 drops of Reagent 1 to an extraction tube.
- Add 3 drops of Reagent 2 to the same tube.
- Place the specimen swab into the tube. Twirl the swab and mix well.
- Wait 1 minute.
- Add 3 drops of Reagent 3 to the same tube. Twirl the swab and mix well.

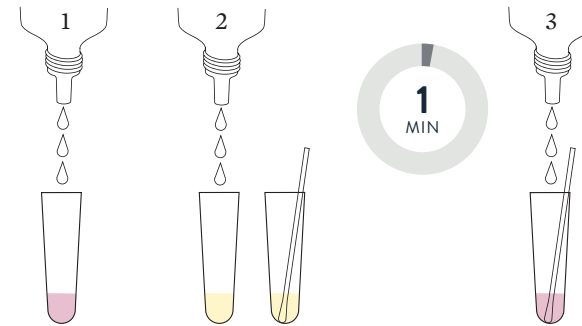


## 2

- Express liquid from the swab by squeezing the tube and rotating the swab as it is withdrawn.
- Discard the swab.
- Apply the dropper tip to the extraction tube.

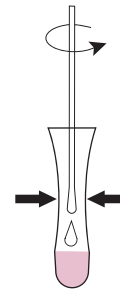

## 3


- Add the entire contents of the extraction tube dropwise into the sample well on the reaction disc.

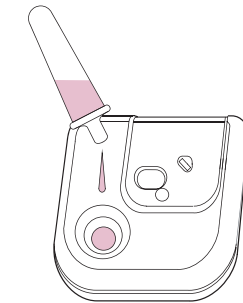

## 4 READ TEST RESULTS

Read the result when a pink/red color appears in the End of Assay window (after approximately 5 minutes).


**Do not interpret the result after 10 minutes from addition of extract.**

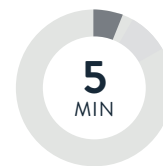


### POSITIVE TEST RESULTS



### NEGATIVE TEST RESULTS

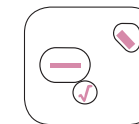

### INVALID TEST RESULTS

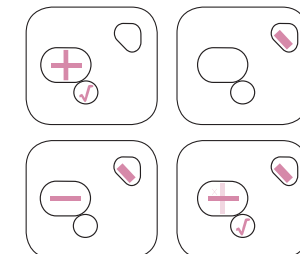

*For more detailed instructions, please refer to the package insert.*