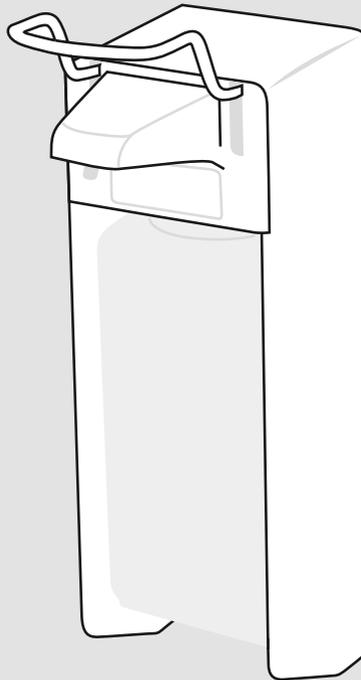
Handbuch

Manual

**AIR-WOLF**

DESINFEKTIONSMITTEL- & SEIFENSPENDER 1,0L  
DESINFECTANT AND SOAP 1,0L

---

Art. 35-450, 35-451, 35-452, 35-453, 35-454, 35-455, 35-456, 35-457, 35-458, 35-459, 35-460, 35-461, 35-462  
Serie Omikron II

## WICHTIG IMPORTANT

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**Read these instructions for use carefully and keep them in a safe place.**

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
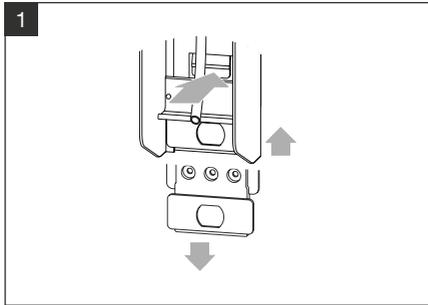
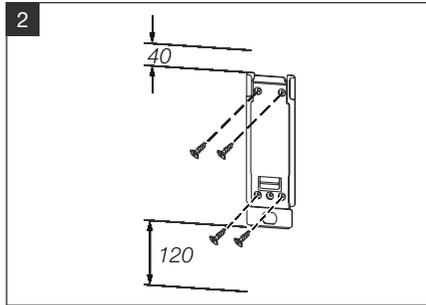
To remove the wall bracket, press the locking catch and slide the wall bracket downwards.

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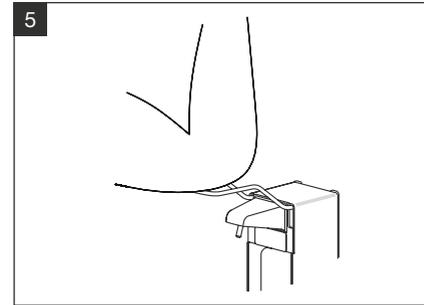
Please take note of the distance above and beneath the dispenser, before fitting it to the wall. Use the enclosed fixing set.

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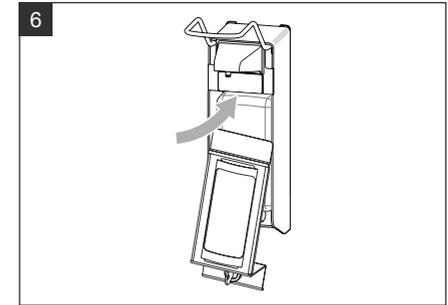
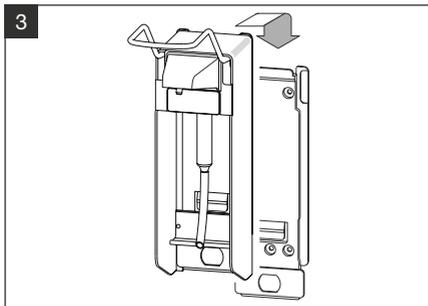
The dispenser and its content can be secured by the optional locking plate.

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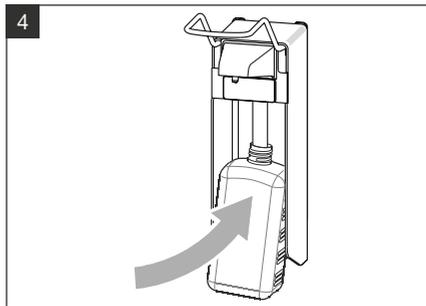
Push the dispenser onto the wall bracket from above. The locking catch will click in and secures the dispenser.

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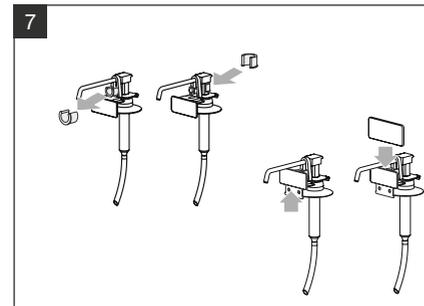
Insert a filled bottle and press the lever a few times until the pump is primed.

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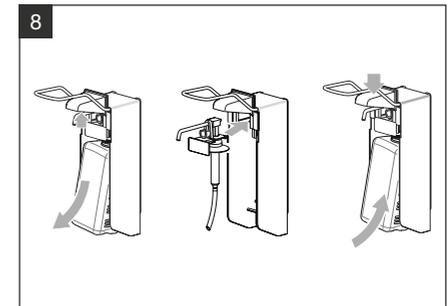
The preset dosage is approx. 1.5 ml/stroke. It can be reduced to approx. 0.9 ml/stroke by inserting the dosage device. If necessary the identification clip can be changed.

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
To change the pump, remove the bottle, slide the outlet cover upwards, then pull the pump out to front. Push the new or cleaned pump into the guide until it clicks into place. Push the spout down again and insert the bottle.

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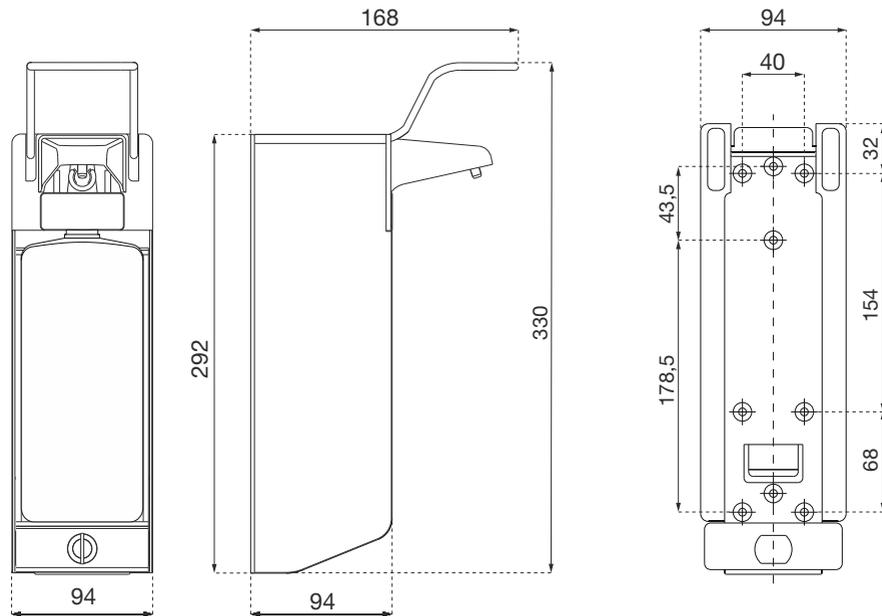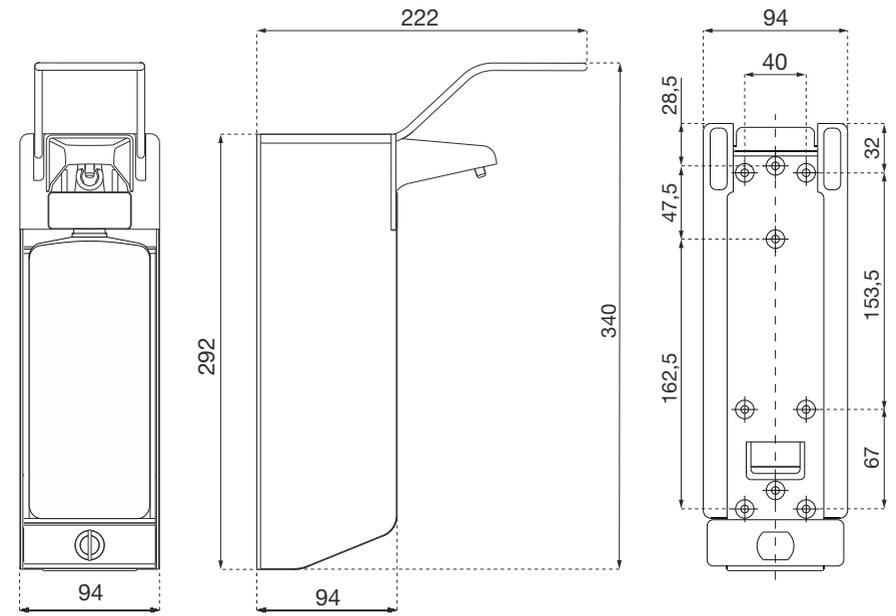**The universal dispenser is a high quality dispenser for dispensing liquid soaps and disinfectants. To guarantee a trouble-free operation, follow the instructions below. Please pass on these user instructions to the maintenance department.**

### Cleaning and Disinfection

The surface of the universal dispenser is made of anodized aluminum, powder-coated aluminum, or brushed stainless steel. Cleaning the dispenser with abrasive detergents or other abrasive materials will damage the surface. It's sufficient just to use damp cloth to clean the dispenser or if necessary, use some warm water. From time to time the pump should be cleaned with hot water. Put some hot water in the bottle and pump it through the dispenser with some force. Please make sure, that all water has been removed from the bottle before re-filling with liquid soap. If necessary the whole dispenser can be cleaned using a good quality disinfectant.

### Liquid Soaps

In general, the dispenser system can be filled with suitable liquid soaps. But, please be aware of the fact, that many products can contain chlorine. Chlorine may corrode the stainless steel pump if the dispenser is not regularly cleaned. The dispenser manufacturer cannot guarantee in this event. To avoid corrosion, it is recommended that the dispenser is cleaned regularly. Make sure that the dispenser is always kept full. This will prevent the remaining soap in the bottle from thickening and clogging the pump. Soap deposits on the outlet and suction valve may occur if the dispenser left unused. This problem will usually resolve itself when the dispenser is used again. In case of stubborn deposits, then it can be cleaned with warm water and a brush. Stubborn deposits should be removed. Keep pumping

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the dispenser until normal dosage has been restored.

### Alcohol Disinfectants

Basically the universal dispenser is suitable for standard alcohol based sanitizers that are available in the market place. Before use, please ensure that they will not act aggressively on the materials of the dispenser. Because of the many and varied compositions used in sanitizers it is advisable to ask the manufacturer's advice in this regard. The information and instructions of the manufacturer must be observed.

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AIR-WOLF GmbH